Instructions for Use Mode d'emploi

elvie™ wave

Additional help and support

If you need any help with your Elvie Wave, please get in touch with our Customer Care team. Visit [elvie.com](https://www.elvie.com) for their details.

To see these instructions in other languages, visit [elvie.com/languages](https://www.elvie.com/languages).

Address

Willow Blossom HoldCo Limited, 9th Floor 107 Cheapside, London, United Kingdom, EC2V 6DN

Actual colours may vary from imagery shown.

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Overview

Safety warnings and precautions

1. Use Product only for its intended use as described herein.
2. This Product is for a single user and should not be shared between users.
3. Only use the Product parts. Do not substitute or modify pump parts.
4. No modification of this equipment is allowed.
5. Inspect the Product prior to each use. Do not use if the Product is damaged or not working properly.
6. Use only with compatible pumps, see global.elvie.com/en-gb/shop/elvie-wave-manual-breast-pump
7. Misuse of the Product may result in personal injury.
8. The Product is not provided sterile and must be cleaned, and the washable parts boiled for 5 minutes before first use.
9. Do not use the Product while driving, when sleepy/drowsy, or while lying down.
10. Care should be taken when storing your Product and associated parts. Improper storage of the Product (e.g., storing wet components, inappropriate temperatures, exposure to sunlight, etc.) may result in damage to the Product and cause harm to the Product user.
11. This Product contains small parts which pose a choking hazard for children under the age of 3. This Product contains tubing which poses a strangulation hazard for children. Keep out of reach of children when not in use. Close supervision is required when used near children.
12. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
13. Monitor milk level during pumping to ensure overflow does not occur.
14. If irritation or discomfort occurs, discontinue use and see a doctor.

Precautions

15. If package appears damaged or tampered with before use, do not use and contact Customer Care via elvie.com/contact.
16. If pumping is uncomfortable: change suction; reposition the Collection Cup; or remove the Collection Cup and discontinue pumping. Consult your healthcare professional.
17. Before each use, visually inspect each part for signs of wear and tear (cracks, chips, tears, discoloration or deterioration).
18. In the event that damage to the device is observed, discontinue use until the parts have been replaced.
19. Before use, ensure that your hands and the breast pump are clean.
20. Use a wash basin dedicated only to cleaning pump parts and infant feeding items.
21. Read all instructions before using the Product.

Symbol definitions

For definition of symbols, please refer to the Symbols Glossary on page 14.

Indications for use

The Product is a manual breast pump to be used by lactating women to express and collect milk from their breasts. The Product is intended for a single user.

When used with the diaphragm assembly and adaptors, the Elvie Wave Collection Cups can be used by lactating women with compatible breast pumps to express and collect milk from their breasts.

Intended use

The Product is intended for lactating women to express and collect milk from their breasts.

Contraindication

No known contraindications.

Usage and operating life

Pump operating life: The pump's operating life is defined as 90 days.

The Collection Cup (Breastshield, Container Back, Gasket, Valve, Container Bowl) are washable parts and should be cleaned after each use.

Product overview

Designed for dual use: Connect to a traditional pump for electronic use or use Handle for wearable manual pumping.

The Collection Cups should be disassembled for cleaning prior to first use, and after each use. The Collection Cups consists of the Flange with Splashguard, Container Back, Gasket, Valve, and Container Bowl. These parts should be replaced every 3 months.

To use your traditional pump, attach the Collection Cups using provided tubing adapters, if applicable. The Elvie Collection Cups are only suitable for double pumping. Use the highest suction, that is still comfortable, on your traditional pump to maximize output.

Manual pumping suction is controlled at the Handle. The switch on the Handle creates two different modes: stimulation (lower suction) and expression (higher suction).

Electric Pumping
with Collection Cups

Manual Pumping
with Collection Cups

Usage instructions

GETTING STARTED: Disassemble Collection Cup

The Product is not provided sterile. Prior to first use, disassemble the Collection Cup into five (5) parts (i.e., Flange with Splashguard, Container Back, Gasket, Valve, Container Bowl) as follows:

1. Detach the Flange Splashguard from the Pour Spout and remove the entire Flange from the container.

2. Separate the Container Back from the Container Bowl (you can use the finger grips on the Bowl or Pour Spout to help).

3. Remove the Valve from the Container Back.


4. Remove the Gasket by carefully lifting it out of the Back Groove.

GETTING STARTED: Clean and sanitize

Disassemble and clean the Collection Cup parts (Breastshield, Container Back, Gasket, Valve, Container Bowl) and then boil for 5 minutes.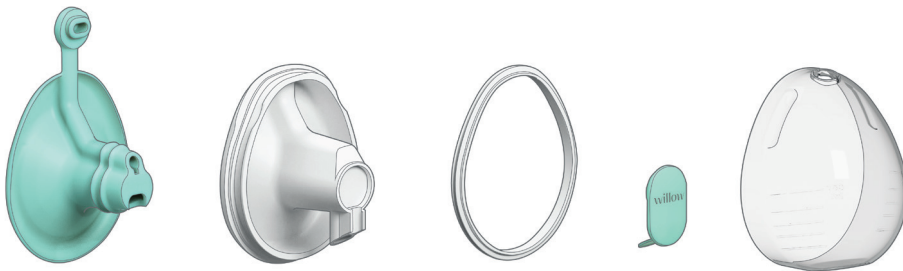

Non-washable parts

Wipe down the outsides of the Handle and outsides of the Tubing with recommended 70/30 isopropyl alcohol. Do not submerge or run under water.

GETTING STARTED: Check your breastshield fit

Find your nipple size by using the nipple sizing tool at elvie.com/size-guide. If your nipple size is less than 21 mm or more than 23 mm, purchase the appropriate sizing insert as indicated on the sizing tool. Proper fit leads to greater comfort.


If you would like assistance with sizing, please contact Customer Care via elvie.com/contact.

GETTING STARTED: Assembly

Note: Wash hands before assembling. Avoid touching the inside of any parts that may come into contact with breast milk to limit risk of infection or contamination.

1. Ensuring the ridges face outward, stretch the Gasket into the Back Groove along the perimeter of the Container Back.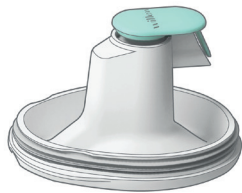

2. Press the Valve onto the Container Back. Ensure it is securely in place.

3. Press the Back into the Container Bowl so it sits flush against the bowl edge.

4. Press the Breastshield into the Container Back.

5. Press the Splashguard into the Pour Spout.

Note: If you need a sizing insert, install the insert into the Breastshield.

ELECTRIC PUMPING: Compatible traditional pumps

Elvie Wave Collection Cups are only compatible with certain Traditional Pumps. Use the provided Tubing Adaptor when applicable.

Ameda™ Mya Joy

Evenflo™ Advanced

Motif™ Luna

Spectra™ S2

Ardo Calypso™

Lansinoh™ Signature

Motif™ Twist

Spectra™ S9+

Baby Buddha™

Medela™ Freestyle

Phillips Avent™ Single Electric

Spectra™ Synergy Gold

Dr. Brown's™ Double Electric

Medela™ Pump in Style

Pumpable Genie™ Advanced

Zomee™ Z2

Elvie™ Stride

Medela™ Swing Maxi

Portable Pump

Spectra™ S1

Elvie™ Stride 2

Motif™ Duo

ELECTRIC PUMPING: Compatibility

There are three adaptors labeled 1, 2 & 3. Some traditional pumps require an adaptor in order to connect tubing to the pump.

ADAPTOR	COMPATIBLE PUMPS
	Baby Buddha™, Motif™ Twist, Zomee™ Z2, Elvie™ Stride, Elvie™ Stride 2
	Medela™ Pump in Style, Spectra™ Synergy Gold, Lansinoh™ Signature
	Medela™ Freestyle, Medela™ Maxi Swing

ELECTRIC PUMPING: Double-cup pumping assembly

The Collection Cups are suitable for double electric pumping only. Do not use the Collection Cups in a single pump configuration with an electric pump. This may lead to discomfort or injury.


When Electric pumping, always use the Diaphragm Assembly as intended - failure to do so may result in milk flowing through tubing to your pump.


Only use electric pumps that are compatible with the Collection Cups. Check compatibility prior to use.

1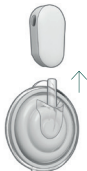

Insert the Diaphragm Assembly into the bottom of the Tube Splitter.

2

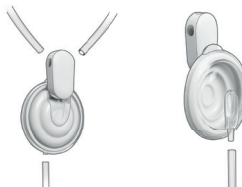Attach the two shorter Tubes to the top of the Tube Splitter and the long Tube to the bottom of the Diaphragm Assembly.

3

Once your Collection Cup is aligned on your breast (see page 10), insert the white Tubing Connector into the Flange Tubing Port on the top of the Flange for both cups.

4


Route the tubing through your shirt and insert the other side of the long Tube into your compatible traditional pump.* Collection Cups are now ready to use!

*use Tubing Adaptor if applicable.

MANUAL PUMPING: Double cup-pumping assembly

Note: Diaphragm Assembly is not needed for manual pumping - keeping the Diaphragm Assembly attached may affect pumping performance.

1

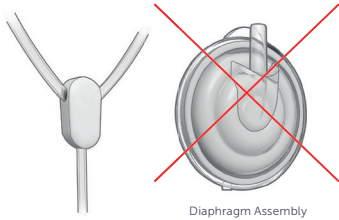Connect Tubing to the Tube Splitter.

2

Once your Collection Cups are aligned on your breast, insert the white Tubing Connector into the Flange Tubing Port on top of the Flange for both cups.

3

Route the Tubing through your shirt and insert the other side of the Tubing into the port of the Handle.

4


Pump is ready to use.

MANUAL PUMPING: Single cup-pumping assembly

Note: Diaphragm Assembly is not needed for manual pumping - keeping the Diaphragm Assembly attached may affect pumping performance.

1

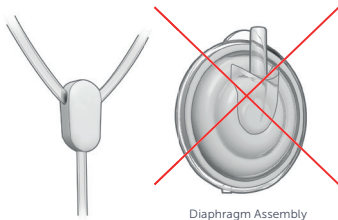Connect Tubing to the Tube Splitter.

2


Plug the white Tubing Connector on the unused short tube into the top center hole of the Tube Splitter. (Make sure it's plugged in securely or suction will not occur).

3

Once your Collection Cups are aligned on your breast, insert the white Tubing Connector into the Flange Tubing Port on top of the Flange for both cups.


4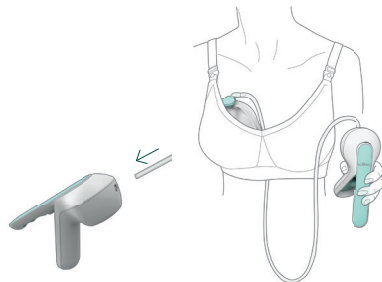

Route Tubing through the bottom of your bra (or however is most comfortable for you) and insert the end of the longer Tube into the port of the Handle.


Aligning the pump

1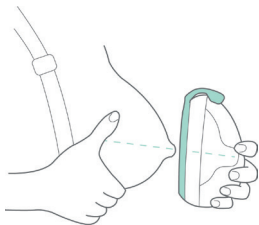


Bring Collection Cup to your breast, centering your nipple in the Breastshield tunnel.

2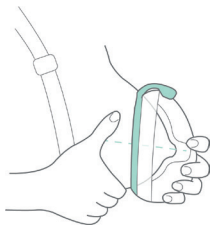

Important: Correct nipple alignment is key to comfort, output and performance.

3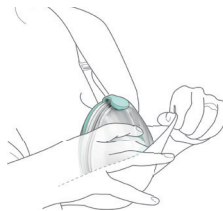

Your bra should be snug and stretchy, with full coverage so that only the top of your Collection Cup peeks out. Fasten your bra to secure the Collection Cup in place.


Creating suction

1


Bring Collection Cup to your breast, centering your nipple in the Breastshield tunnel.

2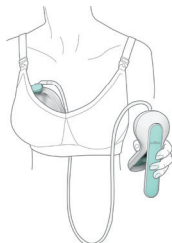

Route Tubing through the bottom of your bra (or however is most comfortable for you) and insert the other side of the Tubing into the port of the Handle. While pumping, keep your upper body in an upright position, whether sitting or standing.

3

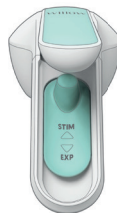It's recommended to start with stimulation mode (switch positioned up), which mimics how a baby feeds, to trigger a letdown. Squeeze and unsqueeze the Handle repeatedly to a comfortable suction level.

4

Once letdown occurs, switch to expression mode by sliding the mode switch inside the Handle down. Continue to squeeze to a comfortable suction level. Expression mode allows for a wider range of suction strength for more efficient milk collection.

Position 1

Position 2

Note: You may notice there is an additional switch on the inside top of the Handle. Ensure the switch is pushed towards 1. This allows for single-cup pumping. Position 2 is not intended for use during single-cup pumping. If you have questions, contact Customer Care via elvie.com/contact.

Transferring milk

Do not pour while tubing is attached to Breast Shield or it may result in milk in tubing.

1. When finished pumping disconnect the Tubing Connector from the Flange Tubing Port.
2. Lean forward slightly and remove the Collection Cup from your bra.
3. While keeping Collection Cup assembled, unplug the pour spout, then slowly transfer milk into a storage bottle or bag.

ROUTINE CLEANING: Washable parts breastshield and collection cup

1. Disassemble the Collection Cup (remember to remove the Gasket from the back too!).
2. Rinse the parts immediately in warm water for 30 seconds (approximately 5 seconds per part).
3. To hand wash the parts:
 - a. Soak with agitation for a minimum of 5 minutes in hot, soapy water. Use mild dishwashing soap. Use a soft-bristled cleaning brush as needed.
 - b. Wash each part for 1 minute in warm soapy water.
 - c. Thoroughly rinse all part surfaces in warm water.
 - d. If any residue is observed, repeat.
 - e. Air dry completely on a clean surface (e.g. clean paper towel or clean drying rack) for a minimum of one 1 hour.
4. Store dry parts when not in use.

Non-washable parts handle and tubing

Do not submerge or run under water

1. Wipe down the Handle and outsides of the Tubing with recommended 70/30 isopropyl alcohol.
2. Inspect the Tubing: if damaged, or signs of mold or other contaminants are present inside the Tubing, discard the Tubing.

If you experience any issues or need help setting up or operating your pump, please visit contact Customer Care via [elvie.com/contact](https://www.elvie.com/contact).

Technical information

Warranty

Elvie Wave is backed by a voluntary limited warranty to the original purchaser against defects in materials and workmanship under normal use for ninety (90) days from the date of delivery of your purchase, as documented by a valid proof of purchase. For the complete terms and warranty information, please visit elvie.com/warranty or contact Customer Care via elvie.com/contact. This policy is offered in addition to your legal rights which can be two years in some countries. Claims must be made within the applicable warranty period. This warranty is valid only in the jurisdictions where Elvie Wave is authorized to be sold and only applies to purchases from Elvie and its authorized distributors and resellers.

This is a single-user product. Use by more than one person may present a health risk and voids the warranty. This product may only be returned for a refund within 30 days of purchase provided it is new, unopened, unused and in its original packaging with the manufacturer's factory seal still intact.

Specifications

Collection Cup dimensions: 121mm x 98mm x 78mm

Handle dimensions: 174mm x 102mm x 73mm

Contents: (2 Collection Cups, Handle, 2 sets of Tubing, 1 Tube Splitter, 1 Diaphragm Assembly, and 3 Adaptors):

See Electric Pump manufacturer technical specification for vacuum strength of compatible pumps.

Vacuum strength: 0 to -245 mmHg

Pump operating life: The pump's operating life is defined as 90 days.

Date of Manufacture: The date of manufacture can be found within the LOT number on the product packaging. The LOT number is written as xxxYYMMDDxxxx, where YY is the last 2 digits of the year, MM represents the month, and DD represents the date of the month. For example, PMF251120 has a manufacturing date of November 20, 2025.

Storage

For storage, store in a room temperature environment.

Disposal information

Dispose of pump parts per your local waste management requirements.

SYMBOLS GLOSSARY

SYMBOL	DEFINITION	Standard from which the symbol was referenced, Reference/ clause/ registration number
	Manufacturer	ISO 15223: 2021, 5.1.1
	Date of Manufacture	ISO 15223: 2021, 5.1.3
	Lot Number	ISO 7000, Reg. no 2492
	Catalog Number; also represents the model number	ISO 15223-1, 5.1.6
	Do not use if package is damaged and consult instructions for use	ISO 7000, 2606
	Consult instructions for use	ISO 15223-1:2021, 5.4.3
	Caution	ISO 15223-1:2021, 5.4.4
	Keep dry	ISO 15223: 2021, 5.3.4
	Keep away from sunlight	ISO 15223-1, 5.3.2

Willow Innovations, Inc.
1975 W. El Camino Real #101
Mountain View, CA 94040

Made in China
www.elvie.com

©2026 Willow Innovations, Inc.

CAREFULLY READ ALL INSTRUCTIONS PRIOR TO USE
Save instructions and sales receipt for future reference.
Issue Date: March 2026

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