How to use

1. Open the bag by removing the hygiene seal and pulling the zipper apart.
2. Pour the milk into the bag and avoid filling past 6 oz/180 ml as breast milk expands when frozen.
3. Remove as much air as possible before sealing the zipper completely.
4. Complete the label at the top of the bag to help track your milk. Ballpoint pens or permanent markers work best, avoid using felt tip markers as they can smudge.

Storing your milk

Lay the bag flat to freeze. To prevent sticking, use paper towels to separate each bag until frozen.

Thawing your milk

Place the sealed bag in warm tap water.

Do not microwave or thaw in hot or boiling water.

Feeding

Transfer the thawed milk into your feeding container. If milk has separated, massage the bag to recombine or mix well once transferred to the container.

Storage location and temperatures

Types of breastmilk	Countertop 25°C or colder (room temperature)	Refrigerator (4°C)	Freezer (-18°C or colder)
Freshly expressed or pumped	Up to 4 hours	Up to 4 days	Within 6 months is best. Up to 12 months is acceptable
Thawed, previously frozen	1-2 hours	Up to 1 day (24 hours)	Never refreeze human milk after it has been thawed
Leftover from feeding (baby did not finish the bottle)	Use within 2 hours after baby is finished feeding		

Precautions and warnings

Date all breast milk and use the oldest breast milk first. DO NOT refreeze thawed breast milk.

DO NOT store breast milk in the freezer door. The door tends to be the warmest part of the freezer and it is coldest towards the back of the freezer.

DO NOT use breast milk if the bag has split or opened in the freezer. The bags are single use, DO NOT reuse them.

Dispose of used plastic bags safely.

Pouch and Breast milk storage bags recyclable

Check with local recycling authority for how to recycle LDPE.

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