Instructions for Use

Thank you for choosing Elvie - we're pumped you're here.

It's our mission to create products that improve women's lives so we hope Elvie Catch makes a difference to yours.

The logo for Elvie Catch, featuring the brand name in a lowercase, rounded sans-serif font. The text is positioned between two horizontal teal bars: a short bar on the left and a long bar on the right.

elvie catch

Language	Page
English	4
Español	6
Svenska	8
Deutsch	10
Français	12
Nederlands	14
Italiano	16
Português	18

For customer care contact details, visit **elvie.com**

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Taking Care of Elvie Catch

Take Elvie Catch apart before cleaning or sterilizing.

Sterilizing (*before first use and subsequently once a day*):

Sterilize parts in boiling water for five minutes.

OR Suitable for microwave, steam or cold water sterilizing methods.

Cleaning (*after every use*):

Wash with clean, soapy water. Rinse thoroughly.

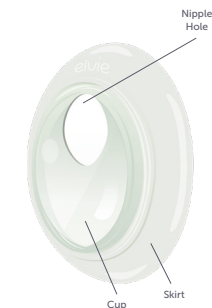
OR Place on the top shelf of the dishwasher.

Elvie Catch should be stored clean, dry and fully assembled.

Using Elvie Catch

1. Push the Cup into the groove on the Skirt. Make sure it is pushed into place all the way around
2. Place your Elvie Catch into your bra, so it's securely held in place. For the best comfort and fit, we recommend wearing Elvie Catch with a nursing bra
3. Center your nipple in the Nipple Hole
4. Be sure to check your milk level regularly, to make sure it doesn't overflow. Do not allow the milk level to go above the bottom of the Nipple Hole
5. We recommend removing Elvie Catch every two to three hours, to allow the nipple to completely air-dry. When removing, it may help to lean forwards slightly
6. Elvie Catch is not designed to store milk. Pour milk through the top of the Nipple Hole as shown in the illustration

Please note: Milk collected over extended periods and left at room temperature for too long should not be fed to your baby.

Safety and Warranty

Intended Use:

'Elvie Catch' is to be used by lactating women to collect leaking milk during and between breastfeeding, when there is unintentional milk flow from the breast.

Materials:

Skirt: Silicone, Cup: Polypropylene

Safety and Warnings:

- Inspect for damage prior to first use and after every use
- Do not store wet or damp parts, as mold may develop
- Do not drop or insert any foreign object into any openings of the device
- If the package appears damaged or tampered with before use, do not use and contact our customer care team
- Keep Elvie Catch away from excessive heat and open flame
- Do not wear Elvie Catch for more than three hours
- Only use Elvie Catch for its intended use as described in this manual

Warranty

Elvie Catch is backed by a voluntary limited manufacturer's warranty against any defects in materials and/or workmanship, when used in accordance with the product instructions: for 90 days from the date of original purchase ("Warranty Period"). This warranty is provided free of charge and is separate to any rights provided under consumer laws or regulations in the country of purchase. It does not exclude, limit or affect any consumer rights or legal claims arising from national laws on the sale of consumer products. This warranty is governed by and constructed under the laws in which the product purchase took place. This warranty does not cover cosmetic deterioration, damage caused by general wear and tear, physical or natural destruction, accident, misuse, modification, neglect or other external causes. This is a single-user product. Use by more than one person may present a health risk and voids the warranty. The warranty does not cover purchases from non-authorized resellers. Any warranty claims must be supported by reasonable evidence, including proof of date of purchase, alongside these warranty conditions. If you experience an issue or identify any defects with your Elvie Catch within the Warranty Period please notify Willow Innovations, Ltd "Willow" at questions@elvie.com. Willow Innovations, Ltd reserves the right to investigate the cause of the defect and may exercise its discretion in determining whether or not to replace the Product or any relevant parts free of charge. In the event that Willow Innovations, Ltd decides to provide a replacement, any replacement will not extend the life of this warranty.

Willow Innovations, Ltd's sole responsibility and liability for any breach of this limited warranty is limited to the replacement of any product that does not comply with the foregoing warranty. Please keep your proof of purchase in the event that you need to contact Elvie customer care for warranty services.

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